

# ICON Frozen-Ambient Combo Shipper

(Qualified for 24 hour transits)



## Pack Ambient Compartment

Central Laboratories



Select Frozen-Ambient Combo Shipper.



Remove foam box lids.



Remove 1 polar pack.



Place 95kPa bag on top of 2 polar packs.



Place polar pack on top of 95kPa bag.



Place foam lid on ambient section.

Instructions for packing the frozen compartment are provided on the following page.



# ICON Frozen-Ambient Combo Shipper

(Qualified for 24 hour transits)



## Pack Frozen Compartment


Central Laboratories



2<sup>nd</sup> foam box is for frozen specimens. Place 2.5 lbs (1.14 kg) of dry ice in bottom. Place up to 3 95kPa bags on ice. Place 4.0 lbs (1.82 kg) of dry ice on top.




Place lid on foam box.

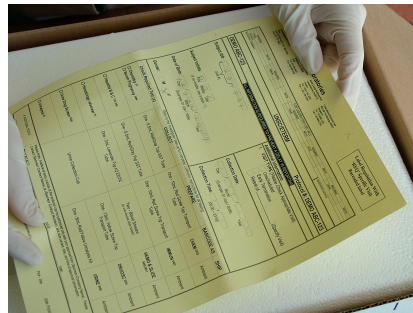


Place yellow copies of reqs. on top.




Fold side flaps on shipper.

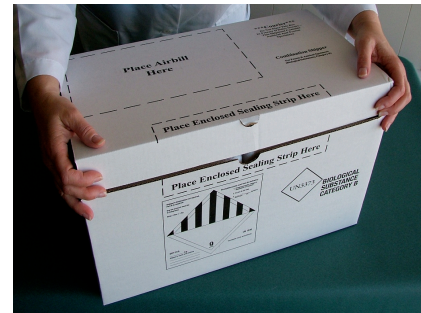


Secure shipper lid.

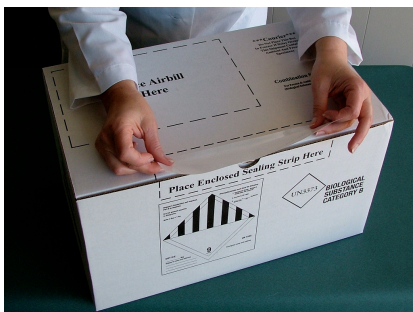

Place adhesive strip on shipper .

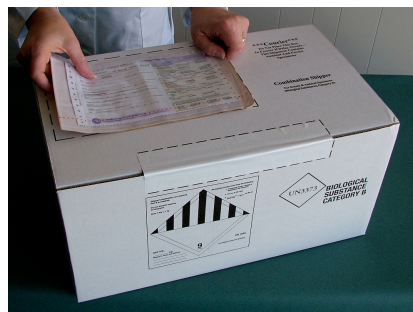

Place completed airbill on shipper.

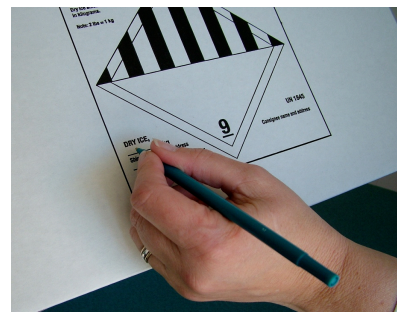

Print dry ice wt. on shipper, as shown.