

ICON Thermal Ambient Shipper



Central Laboratories

Note: The Thermal Ambient Shipper box is sent to sites unassembled along with the required gel wraps. Use the following instructions to assemble the shipper box prior to use.



Obtain unassembled box and 2 gel wraps.



Turn box over to reveal inside of box.



Fold inner flaps to form rectangle.



Fold down outer flaps on each side.



Secure outer flaps into box slits.



View of box when assembly is complete.

The following steps demonstrate the proper method for packaging specimens in the Thermal Ambient Shipper.



Select assembled box and 2 gel wraps.



Place 1st gel wrap in box as shown.




Place 2nd gel pack over 1st gel wrap..



Place 95kPa specimen bag as shown.

Continued on next page

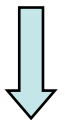

ICON Thermal Ambient Shipper (Continued)



Central Laboratories



Fold one gel wrap over 95kPa bag.




Fold 2nd gel wrap over first gel wrap.



Gel wraps should fully cover 95kPa bag.




Secure shipper lid.

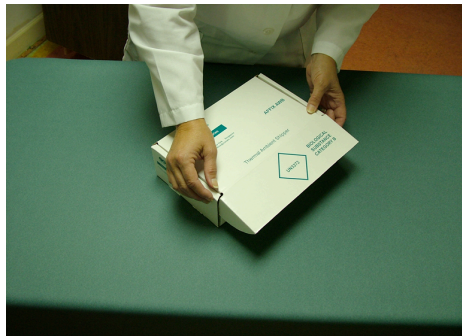

Fold lid flaps into box front .



Shipper box with fully secured lid.

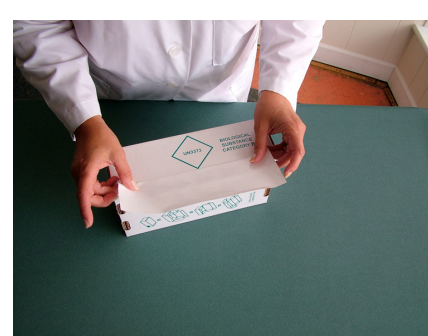

Place adhesive strip on shipper.



Place completed air bill on shipper.