

## Packing Instruction 650 for UN3373 Category B Specimens (Refrigerated)



1. Components.



2. 95 Kpa Bio-Bag & Absorbent Pouch.



3. Place Tubes/Vials into the absorbent Pouch



4. Place Absorbent Pouch (Containing tubes). Into the 95 Kpa Bio-Bag.



5. Peel the brown strip & seal Bag.



6. Place Sealed Bio-Bag in the Outer box. Seal with security strip provided



7. Insert the two pre-conditioned 'Cool-Packs' in the Thermo Unit.



8. Place specimen box in the centre of the thermal unit and between the two Cool-Packs. Place lid and fold over all the outer flaps.

**Your Refrigerated Category B Specimen is now ready for Transportation**